



1 FIRST STEP

2 NEEDLE CAP DETACHMENT WITH ONLY ONE HAND




3 MEDICINE ASPIRATION

4 NEEDLE DETACHMENT WITH ONLY ONE HAND



5 NEEDLE DISPOSAL

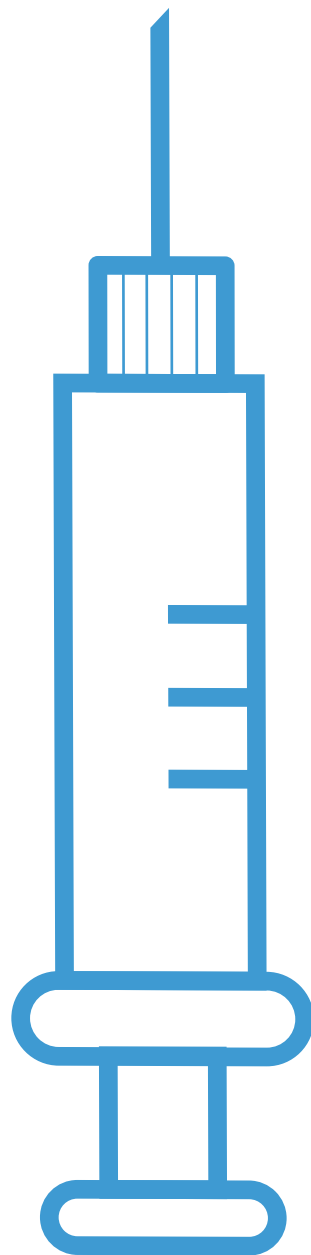

Aghetto srl
PAVIA - ITALY
info@aghetto.com
www.aghetto.com

Aghetto 
A NEEDLE REVOLUTION



**THE NEW SYRINGE
USABLE WITH
ONLY ONE HAND**

*Easy
and Safe*



THE PROBLEM

With currently commercially available syringes the detachment of the needle-cap and of the needle is not easy and it is not safe, there is a risk of needlestick injuries, and two hands are required but operators often don't have both hands free.

THE SOLUTION

Aghetto is an innovative syringe that allows the operator to detach the needle-cap and the needle easily and safely with only one hand.

The other hand is free and can be used for other tasks (e.g. holding a medicine vial, a cotton swab, a venous cannula, other medical tools or - in veterinary medicine - to keep an animal still).



Aghetto is recommended for doctors, nurses, veterinarians, lab technicians and anyone using syringes including those who for health reasons cannot use both hands.

EASY AND SAFE

Aghetto is:

EASY TO USE

just turn a tiny wheel

UNIQUE

there is no similar device on the market

**PATENT
PROTECTED**



Aghetto facilitates the preparation of therapies:

- the operator can detach the cap from the needle with only one hand and hold the drug vial with the other hand
- the operator can detach the needle with only one hand and replace it with the other hand

Aghetto facilitates the administration of therapies:

- the operator can detach the cap from the needle with only one hand and hold the disinfectant cotton swab with the other hand
- the operator can detach the needle with only one hand and connect the venous access with the other hand

WITH ONLY ONE HAND



Aghetto is very useful in emergency care:

- the operator can detach the cap from the needle and the needle from the syringe easily and quickly with only one hand and handle other tools with the other hand

Aghetto is very useful in ultrasound-guided procedures:

- the operator can detach the cap from the needle and the needle from the syringe with only one hand and handle the ultrasound probe with the other hand

Aghetto is very useful in laboratory medicine:

- the operator can detach the cap from the needle and the needle from the syringe with only one hand and handle other tools (e.g. test tube) with the other hand

Aghetto is very useful in veterinary medicine:

- the operator can detach the cap from the needle and the needle from the syringe with only one hand and hold still the animal with the other hand

CORRECT USE OF SHARP AND PUNGENT WASTE CONTAINERS:

Aghetto allows the operator to detach the needle easily and safely from the syringe with one hand and dispose of the needle only in the container and not the whole syringe as often happens.

